
# QUICKLOOP® HOME CARE INSTRUCTIONS

## DRESSING CHANGES

1. Change the dressing and wash the incisions with soap and water 2-3 times per day. Additional dressing changes may be required if the gauze bandage becomes saturated with pus or blood. Simply peel the dressing and tape away from the skin and discard.
2. Make sure the tubing moves freely and does not become stuck. Grab the hub and bring it close to one of the incisions. Then move the tubing back and forth.
3.  **YES**    **NO** Your provider would like you to irrigate the abscess cavity during dressing changes. To irrigate:
  - a. Fill the syringe provided by your health care provider with \_\_\_\_\_
  - b. Attach the syringe to the hub by twisting it clockwise.
  - c. Move the hub toward the incision until the irrigation holes are under the skin (as shown in the picture).
  - d. Press down on the plunger until all the liquid has flowed into the abscess cavity.
  - e. Repeat steps (a – d) 2 more times.
  - f. Detach the syringe from the hub by twisting counterclockwise.
4. Re-adjust the tubing so the irrigation holes are outside the skin.
5. Re-dress the incision by placing one gauze pad between the hub and skin and secure with tape. Use another gauze pad (or several) to cover the incisions and the device.
6. Repeat the steps above 2-3 times per day until the device is removed or as directed by your provider. Once drainage has stopped, cover device loosely to allow area to remain dry.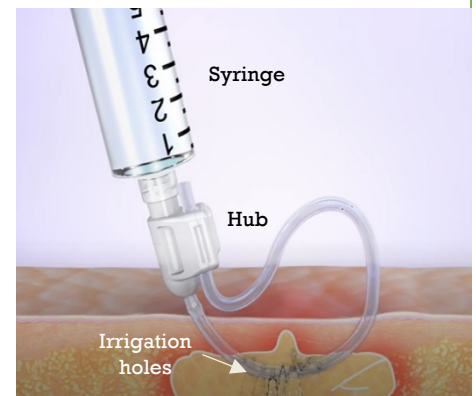

## DEVICE REMOVAL

- Follow-up with a medical professional for device removal**

OR

- Remove device at home**

1. If instructed to remove device at home, the device is ready for removal when all of the following are true:
  - ✓ At least three days have passed since device insertion, up to \_\_\_\_\_ days as directed
  - ✓ There is no pus or blood draining from either incision
  - ✓ There is very little redness and swelling around the incisions
  - ✓ There is very little pain at the site of the procedure
2. To remove the device, hold the plastic hub with one hand and use your other hand (or another person) to cut one side of the loop. The device should then easily slide out. **Be sure not to cut both sides of the loop.**
3. Discard the device. If any piece of the device remains inside the wound, please see a medical professional.

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